

Meril

Endo-Surgery

Topical Skin Adhesive
MERIGLUTM
N-Butyl Cyanoacrylate



MERIGLUTM Topical Skin Adhesive is a waterproof formula consisting of monomeric N-Butyl Cyanoacrylate, which polymerises quickly when comes in contact with tissue fluid.

MERIGLUTM is intended for topical application to close easily approximated skin edges of wounds from surgical incisions



FAST



RELIABLE



SECURE

**WE GOT YOU
GLUED to the
best outcomes.**

MBSTM
Meril Biosurgical Solutions

Advantages

Superior Strength and Flexibility

Faster Set time < 50 seconds

Easy Application

Patient Comfort and Appreciable Cosmesis

Effective Microbial Barrier

Application Instructions

Opening of MERIGLU™ pouch:

Pull apart pouch to expose the sterile, single patient use tube and tip.

Hold tube with the tube opening pointing upward and lightly fix the tip on it.

Once the tip is fixed tightly, the tube is ready for use.

Preparation of wound:

Dry the moist wound prior to application, cleaning excessive blood and debris.

Approximate the tissue edges and apposition during application.



Process:

Lightly hold the wound together for about 30-40 seconds to allow MeriGlu to settle.

Subcuticular sutures may be used prior to application of MeriGlu for securing wounds under tension.

Application of MERIGLU™:

Apply light pressure to the tube to expel MeriGlu from the sterile tip.

MeriGlu is to be applied as a very thin film along the edges of the approximated wound, and not directly inside the wound.

Ordering Information

Product Code	Product Content	Ordering Qty.
BCA-0.5	MeriGlu - 0.5 ml	6 tubes/ box
BCA-01	MeriGlu - 1 ml	

Usage Indications

Skin Closure for small to medium incisions (Upto 6 inches) in surgeries of

- General surgery – laparoscopy, hernia, appendicitis, vascular surgery
- Plastic surgery – cleft lip, excision of scars, skin graft
- Gynecology – caesarean section, hysterectomy, tubectomy
- Orthopedic – tendon repair, bone graft, cartilage binding
- Surgical oncology – thyroid and breast malignancy surgery
- Pediatrics – management of pediatric lacerations
- Dental surgery
- ENT surgery
- Trauma Care



MERIL ENDO-SURGERY PVT. LTD.

Corporate Headquarters, Vapi

E1-E3, Meril Park, Survey No. 135/2/B & 174/2, Muktanand Marg, Chala, Vapi- 396 191, Gujarat, India.

Tel.: +91 260 3052 100, Toll Free No.: 1800-4194-433

India Sales and Marketing, Mumbai

601-612, Midas, Sahar Plaza Complex, M.V. Road, J.B Nagar, Andheri (East), Mumbai 400 059, India. Tel.: +91 22 307 32 400

Email: endosurgery@merillife.com, Web: www.merillife.com